

PACS WEB PORTAL Instructions:

1. Open a browser,
Go to Address/URL: mmmh.gomedsys.com
Enter User Name and Password and click login

PACS PORTAL

MEDSYS 2014 (c) KCC

USER LOGIN
for MEDSYS Applications and Doctors

8

.....

Remember Me

LOGIN

MAMCRAJ REGIONAL HOSPITAL MEDICAL
we care since 1963

2. Search for patient
 - >Check 'Your Reading only' if you want to see only readings assigned to you. Uncheck to see all studies.
 - >Uncheck 'Pending Results' to include studies with results in the list, check to see studies without any results yet
 - >Click on the Patient Name to open the image

gomedsys.com/pacs/pacsresults.aspx

Dr. [Name] Doctor Click your name to update your default dept. and mobile #

PACS Results Download Web Viewer Application (2.5 MB). [Dicom File Viewer] Instructions

Patient ID/HospNum [] Accession / Request # [] Radiologist MD83036

Patient Last Name [] First [] Your Reading only

Study Date 01/26/2017 Today Clear Pending Results

Modality [] All CR DX MG CT MR US NM DR

Search

PatientID	Patient Name	Birthdate	Gender	Study Date	Modality	Study Desc.	AccessionNumber	ReferringPhysician	ResultStatus
31425	CO	12/29/1957	M	01/26/2017 6:12PM	CR	HEART & LUNGS (1 VIEW) ADULT	1032589	[]	
42154	CUR	09/30/1970	M	01/26/2017 5:54PM	CR	HEART & LUNGS (1 VIEW) ADULT	1032590	A' JONES	
113208	GUI	03/12/1936	F	01/26/2017 5:27PM	CR	ADULT CHEST - 1 VIEW		[]	
40539	PER	11/25/1962	F	01/26/2017 5:21PM	CR	ADULT CHEST - 1 VIEW		UC	
94113	ANCI	08/24/1985	M	01/26/2017 5:15PM	CR	PELVIS - 1 VIEW		T	
94113	AN	08/24/1985	M	01/26/2017 5:11PM	CR	'SKULL - 2 VIEWS'		T	
94113	ANCHI	08/24/1985	M	01/26/2017 5:09PM	CR	ADULT CHEST - 1 VIEW		T	

3. Enter the Result

- > Enter the Result in the Box or search for a template (under Templates., enter the search word and click Search. Then click Select to use that template)
- > Enter Save as Draft or Save as Verified. Once Verified, the result cannot be altered using the Web Interface.

Dr. [Name] Doctor
Click your name to update your default dept. and mobile #

PACS Results [Download Web Viewer Application \(2.5 MB\)](#) [\[Dicom File Viewer\]](#) [Instructions](#)

Search		History:						
PatientID	Patient Name	Birthdate	Gender	Study Date	Modality	Study Desc.	AccessionNumber	ReferringPhysic
594113	A	08/24/1985	M	01/26/2017 5:15PM	CR	PELVIS - 1 VIEW		
594113	A	08/24/1985	M	01/26/2017 5:11PM	CR	'SKULL - 2 VIEWS'		
594113	AP	08/24/1985	M	01/26/2017 5:09PM	CR	ADULT CHEST - 1 VIEW		

ANURAG MOHANTY, (594113) 01/26/2017 5:15PM Gender: M PELVIS - 1 VIEW () ResultStatus: Modality:CR

Plain Text

no result

- [Save as Draft \(F\)](#)
- [Save as Reported \(O\)](#)
- [Save as Verified \(V\)](#)

Templates...

Section pelv [Clear Section](#) [Cancel](#)

Search: [Search](#)

Select Exam_ID	examname	Title	Preview	Select Exam_ID	rdname
01083	PELVIC		Preview	Select	E

SeriesNum	SeriesDescription	Modality	Load Images/Result	View	ActiveX View	Window
1		CR	Per Image	All Images in this Series	ActiveX	View New Window

- 1 + Contrast - 1 +

