


## PACS WEB PORTAL Instructions:

1. Open a browser,  
Go to Address/URL: [pms.h.gomedsys.com](http://pms.h.gomedsys.com)  
Enter User Name and Password and click login

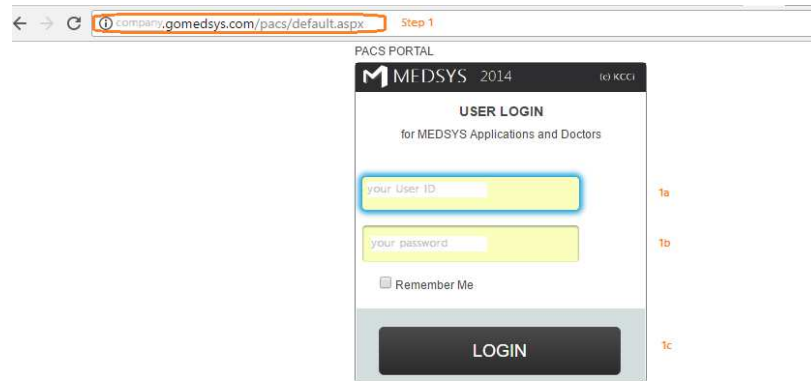

2. Search for patient
  - >Check 'Your Reading only' if you want to see only readings assigned to you. Uncheck to see all studies.
  - >Uncheck 'Pending Results' to include studies with results in the list, check to see studies without any results yet
  - >Click on the Patient Name to open the image

gomedsys.com/pacs/pacsresults.aspx

Dr. [Name] Doctor Click your name to update your default dept. and mobile #

PACS Results Download Web Viewer Application (2.5 MB). [Dicom File Viewer] Instructions

Patient ID/HospNum [ ] Accession / Request # [ ] Radiologist MD83036

Patient Last Name [ ] First [ ]  Your Reading only

Study Date 01/26/2017 Today Clear  Pending Results

Modality [ ] All CR DX MG CT MR US NM DR

**Search**

PatientID	Patient Name	Birthdate	Gender	Study Date	Modality	Study Desc.	AccessionNumber	ReferringPhysician	ResultStatus
131425	CO	12/29/1957	M	01/26/2017 6:12PM	CR	HEART & LUNGS (1 VIEW) ADULT	1032589	DR. [Name]	
142154	CUR	09/30/1970	M	01/26/2017 5:54PM	CR	HEART & LUNGS (1 VIEW) ADULT	1032590	A. [Name]	
113208	GUI	03/12/1936	F	01/26/2017 5:27PM	CR	ADULT CHEST - 1 VIEW		[Name]	
140539	PER	11/25/1962	F	01/26/2017 5:21PM	CR	ADULT CHEST - 1 VIEW		UC [Name]	
194113	ANCI	08/24/1985	M	01/26/2017 5:15PM	CR	PELVIS - 1 VIEW		T. [Name]	
194113	ANCI	08/24/1985	M	01/26/2017 5:11PM	CR	'SKULL - 2 VIEWS'		T. [Name]	
194113	ANCHI	08/24/1985	M	01/26/2017 5:09PM	CR	ADULT CHEST - 1 VIEW		T. [Name]	

3. Enter the Result

- > Enter the Result in the Box or search for a template (under Templates., enter the search word and click Search. Then click Select to use that template)
- > Enter Save as Draft or Save as Verified. Once Verified, the result cannot be altered using the Web Interface.

Dr. [Name] Doctor [Click your name to update your default dept. and mobile #](#)

PACS Results [Download Web Viewer Application \(2.5 MB\)](#) [\[Dicom File Viewer\]](#) [Instructions](#)

Search		History:						
PatientID	Patient Name	Birthdate	Gender	Study Date	Modality	Study Desc.	AccessionNumber	ReferringPhysic
594113	A	08/24/1985	M	01/26/2017 5:15PM	CR	PELVIS - 1 VIEW		
594113	A	08/24/1985	M	01/26/2017 5:11PM	CR	'SKULL - 2 VIEWS'		
594113	AP	08/24/1985	M	01/26/2017 5:09PM	CR	ADULT CHEST - 1 VIEW		

ANURAG MOHANTY, (594113) 01/26/2017 5:15PM Gender: M PELVIS - 1 VIEW ( ) ResultStatus: Modality:CR

Plain Text

no result

Save as Draft (F)

Save as Reported (O)

Save as Verified (V)

Templates...

Section pelv [Clear Section](#)

[Cancel](#)

Search: [Search](#)

Select Exam_ID	examname	Title	Preview	Select Exam_ID	rdname
01083	PELVIC		Preview	Select	E

SeriesNum	SeriesDescription	Modality	Load Images/Result	View	ActiveX View	Window
1		CR	Per Image	All Images in this Series	ActiveX	View New Window

- 1 + Contrast - 1 +

Small 100% Original

