



How to Install Medsys PACS


Step 1. Access **192.168.100.1** in run command and copy the folder "**copylocal**" to local disk.

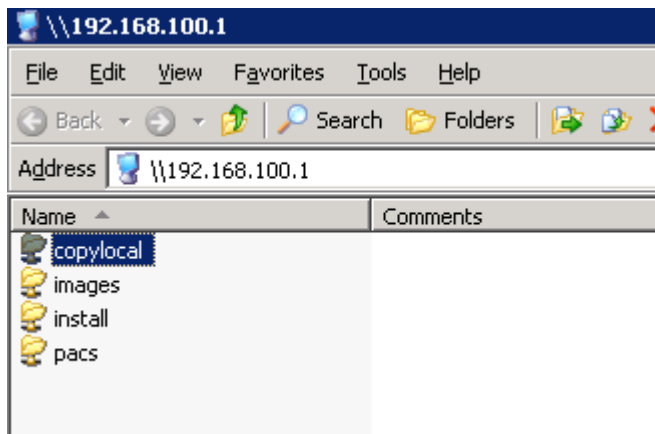


Step 2. Open the copied folder "copylocal" on your local disk and run **Setup.bat** located on copylocal\MEDSYS\RUN\dlls.

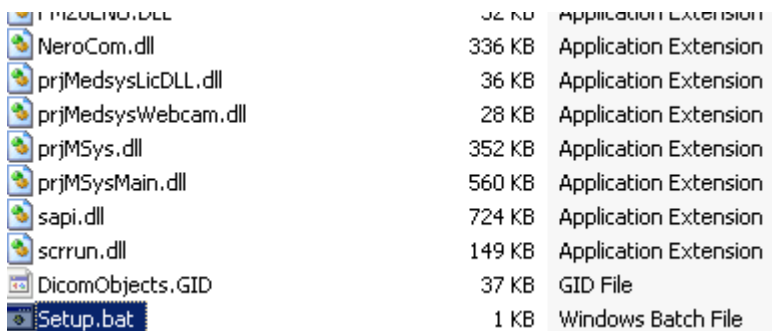

Step 3. Access **192.168.100.1** in run command and open the folder "**pacs**".

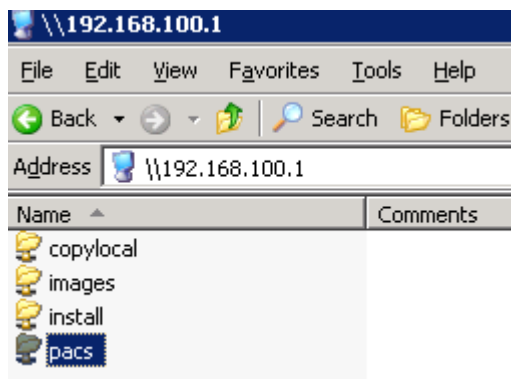


Step 4. Register **PACS_DLL.dll** by double-clicking it.

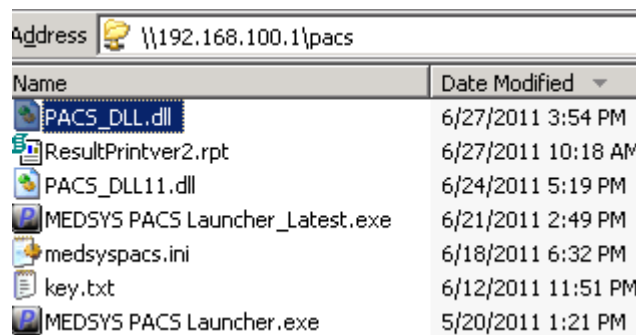


Name	Date Modified
PACS_DLL.dll	6/27/2011 3:54 PM
ResultPrintver2.rpt	6/27/2011 10:18 AM
PACS_DLL11.dll	6/24/2011 5:19 PM
MEDSYS PACS Launcher_Latest.exe	6/21/2011 2:49 PM
medsyspacs.ini	6/18/2011 6:32 PM
key.txt	6/12/2011 11:51 PM
MEDSYS PACS Launcher.exe	5/20/2011 1:21 PM


Check the box **"Always use the selected program to open this kind of file."** Click on **"Browse"** button.

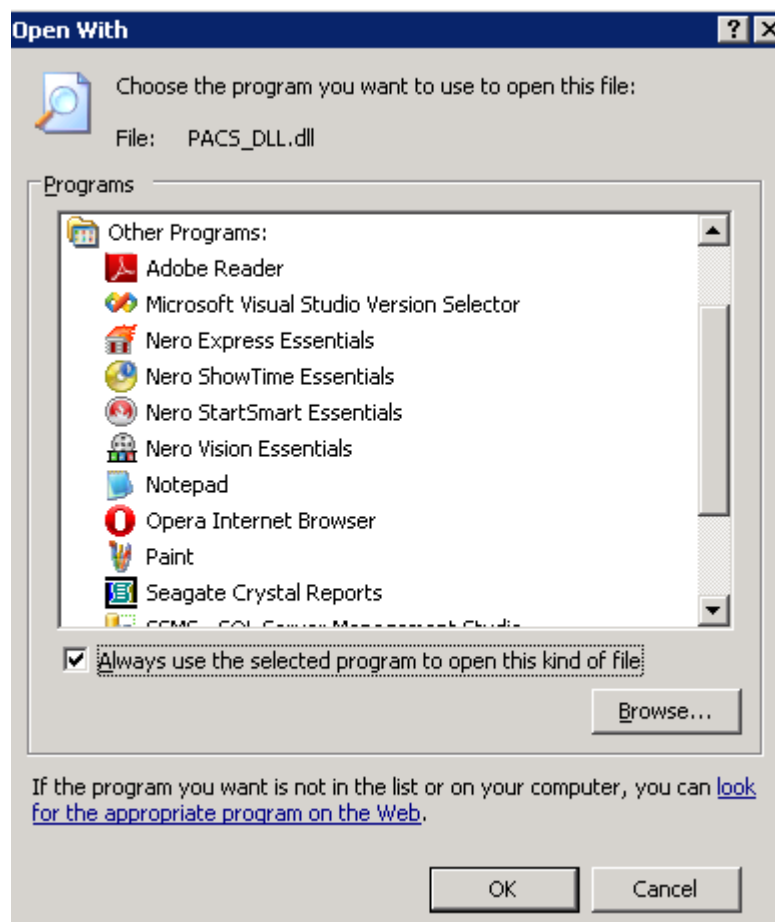


Go to **c:\windows\system32**. Look for **regsvr32.exe** and open it by double-clicking.

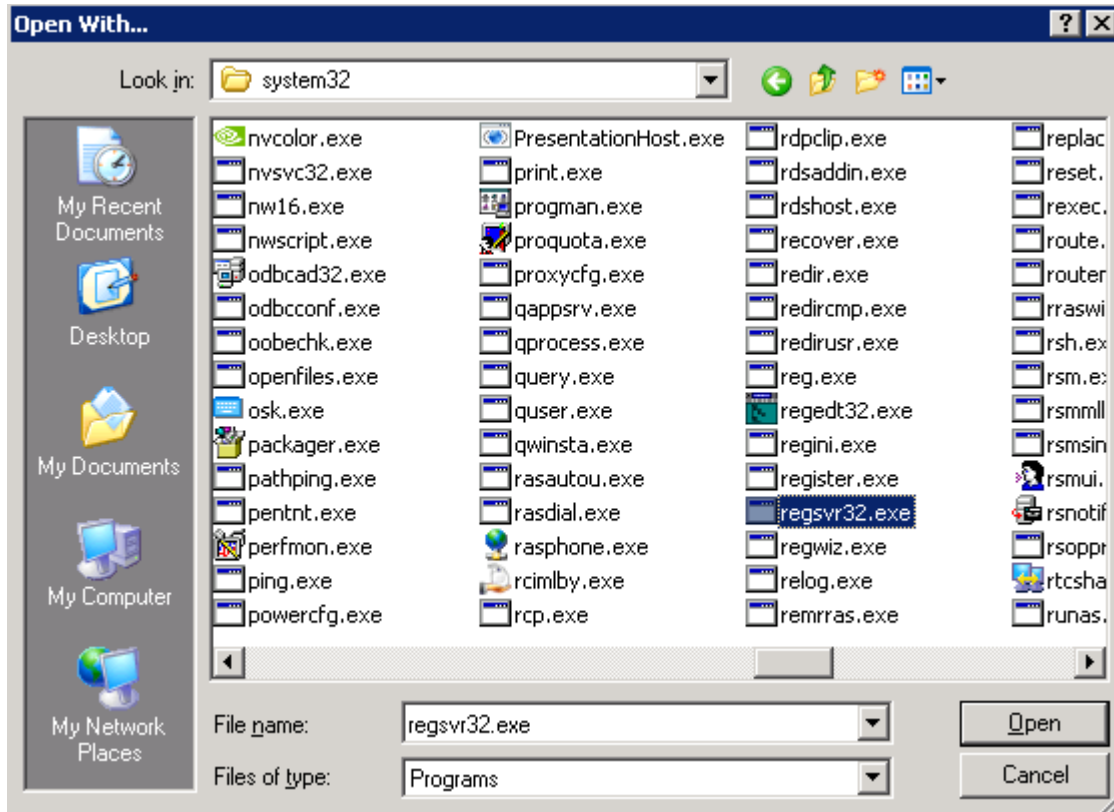


Notice the **Microsoft(C) Register Server** added on the list of programs. When you open or register dll file next time, it will automatically choose by default the Microsoft Register Server. Click on **OK** button to register the PACS_DLL.dll.

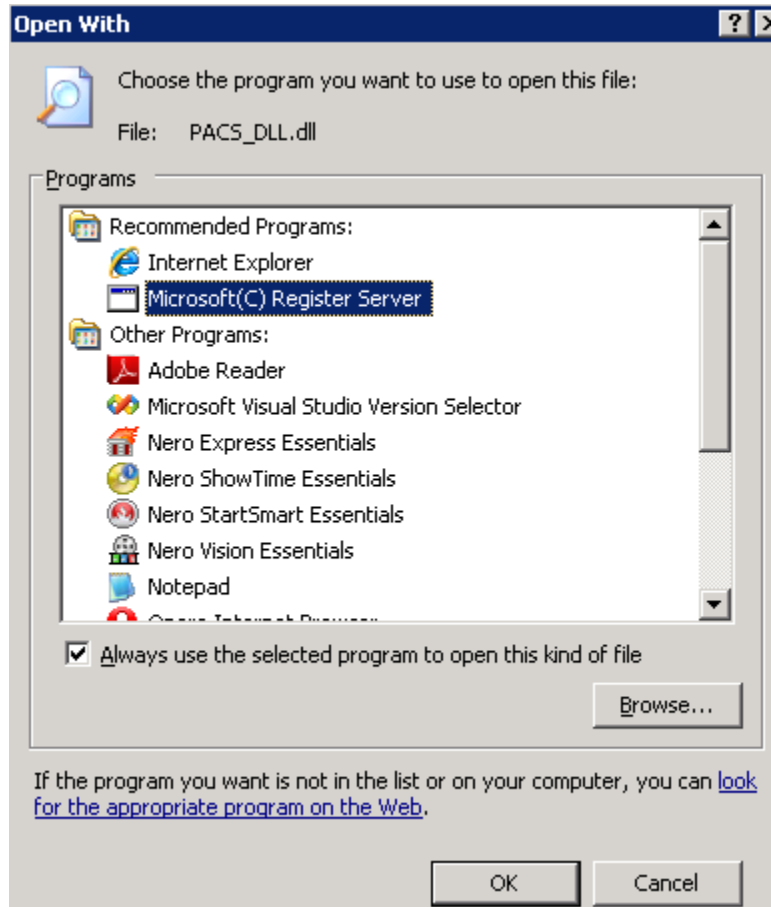

A window will appear if you successfully registered PACS_DLL.dll.

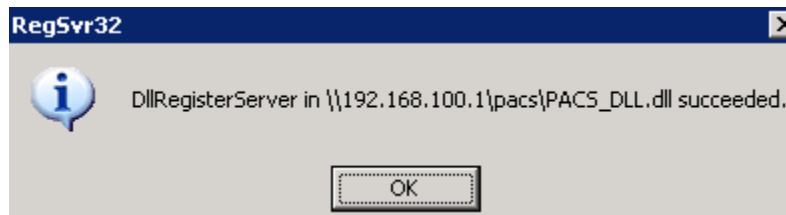

Step 5. Look for **MEDSYS PACS Launcher.exe** and make a shortcut on desktop by right-clicking on it.

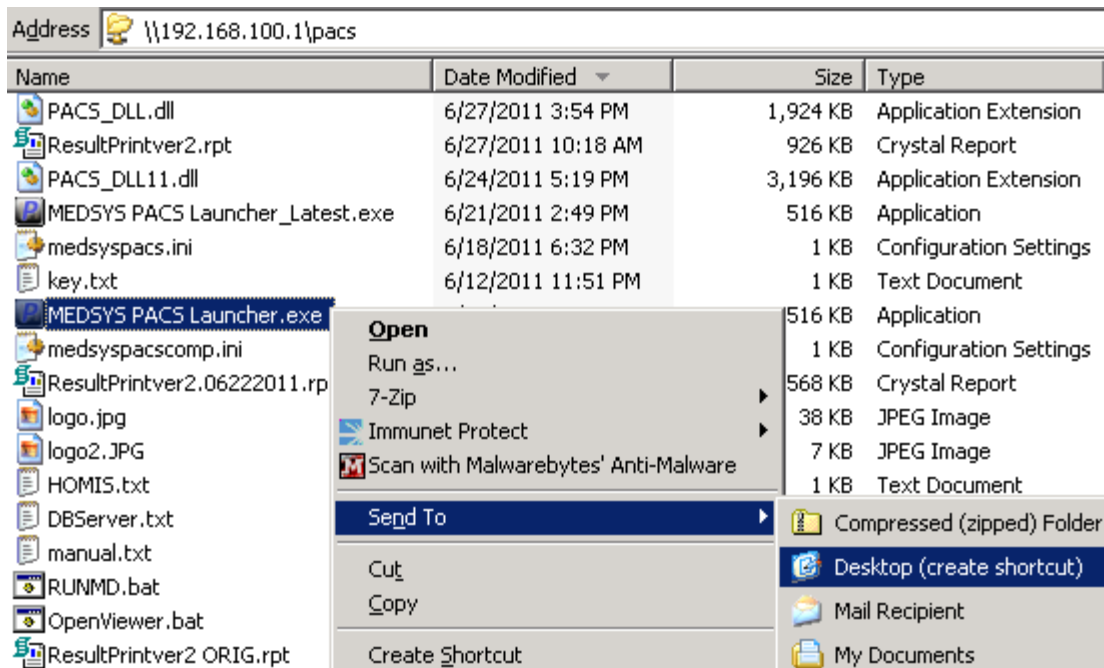

Step 6. Go to your desktop and open **Medsys PACS Launcher icon**.




The first time you open it, it will require the PACS server admin account.

Username: administrator


Password: a1234(shift)5678

Check the box "**Remember my credentials**" to connect it automatically next time you open it.

Step 7. Register Medsys Software.

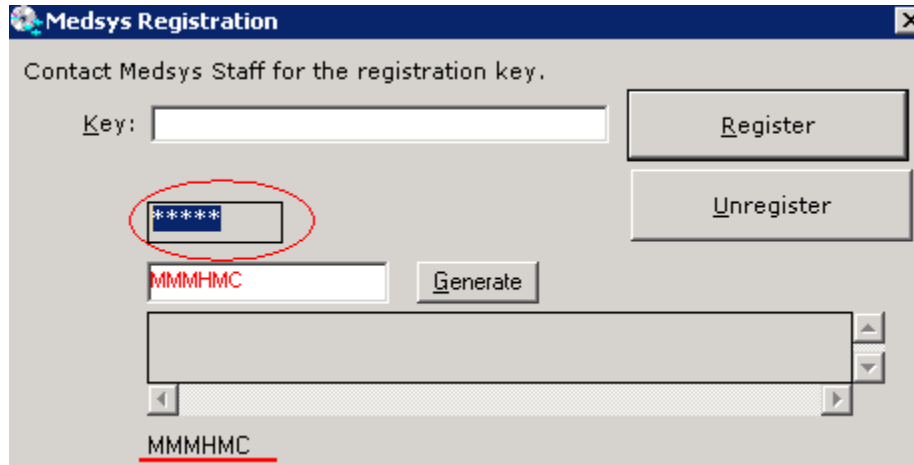

Type on the box (circled on red) **med@k**. Type **MMMHMC** on the box before Generate button. Click on **Generate** button to generate software key.

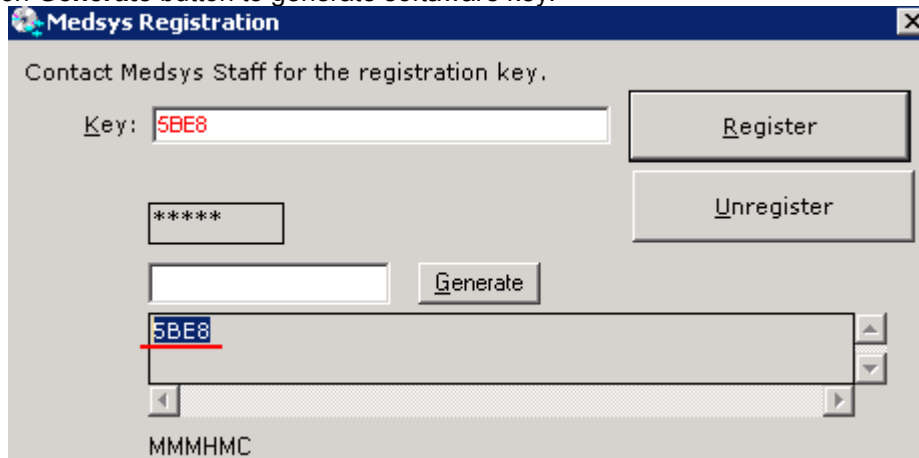


Copy the generated characters and paste it on the **Key** box. Click **Register** button afterwards. A small window will appear saying that it was registered successfully.

Step 8. Setup up connectivity.




Hit **(space)medsys** on keyboard under the **Password** box. Fill in the Server Login.
Server: MEDSYS-PACS\PACS or 192.168.100.1\PACS
Database: PASSWORD
User: sa
Password: pacs

Click the **Test Connection** button to test the connection.

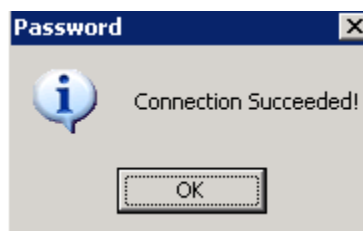

Connection Succeeded window will appear if connected successfully. Otherwise, Error Accessing Server will appear if not connected successfully. Double-check the entries on Server Login to see if the entered data are correct.

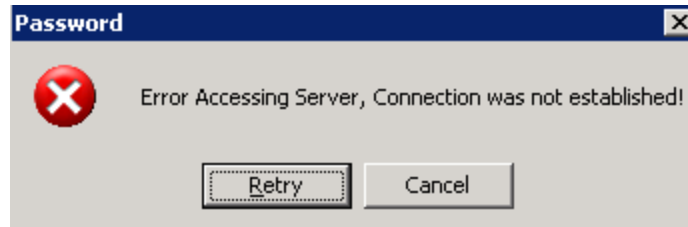

Step 9. Try to login using your user account. The installation is successful once you have logged in successfully.

Notes:

1. Administrator account should be enabled.

To do this, right-click on **My Computer** and select **Manage**.

Look for **Local Users and Groups**. Go to **Users** and search for **Administrator** account. Right-click it and go to **Properties**.

Uncheck **Account is disabled** button then click OK.

2. Medsys Launcher exe should be executable.

To do this, go to run command and type **gpedit.msc** to go to Group Policy Object Editor.

Under **User Configuration**, go to **Administrative Templates -> Windows Components ->**

Attachment Manager.

Look for the setting **Inclusion list for low file types**. Right-click it and go to **Properties**.

Select **Enabled** and type **.exe** on the **Specify low risk extension files**.

Click **OK** to save changes.